

# Acucy® Influenza A&B Test

## HELPING YOU HIT THE MARK WITH INFLUENZA TESTING

The **Acucy® Influenza A&B Test** on the Acucy® System provides clinicians flexibility in workflow and accurate, standardized results for improved patient care.

### Features and Benefits


- **Accurate** - Best-in-class influenza A test performance\*
- **CLIA Waived** - Easy to use
- **Standardized digital results on a reader**
- **Flexible** - Read Now and Walk Away modes
- **Exceptional customer service** - Online training modules and technical support staffed by certified medical technologist/lab professionals

### Performance Characteristics


	Sensitivity	Specificity	Sample Type	Limit of Detection**	Time to result
Type A	96.4%	96.0%	Nasal/Nasopharyngeal	1.4 x 10 <sup>1</sup> TCID <sub>50</sub> /mL	15 minutes
Type B	82.3%	98.1%		2.4 x 10 <sup>1</sup> TCID <sub>50</sub> /mL	

\*When compared to immunoassay products in market

\*\*See IFU

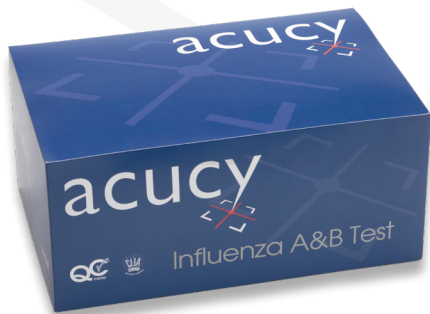

### SENSITIVITY

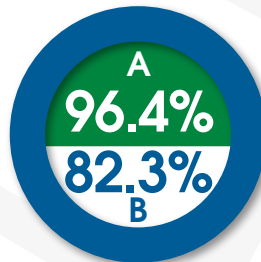

### SPECIFICITY




# Acucy® Influenza A&B Test

## SIMPLE TO USE / RESULTS IN MINUTES


1

EXTRACT PATIENT SPECIMEN

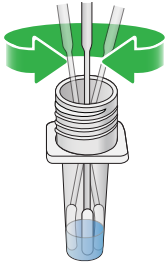

2

SQUEEZE SIDES OF VIAL AND PULL OUT THE SWAB TO REMOVE EXCESS LIQUID

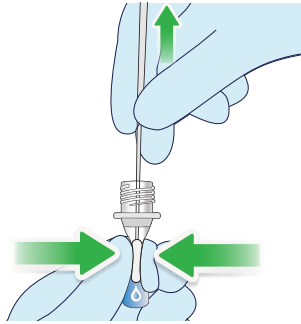

3

ADD DROPPER TIP



4

ADD DROPS INTO CASSETTE



## RUN TEST

### WALK AWAY/NORMAL MODE:

Reader will time test and display results

- Remove cassette from foil pouch
- Place cassette in Reader
- Add sample to cassette
- Close reader drawer
- Reader will time test for 15 minutes
- Reader displays results

### READ NOW MODE:

Test timed outside of the Reader

- Remove cassette from foil pouch
- Add sample to cassette
- Set timer for 15 minutes
- Place cassette into Reader
- Close Reader drawer
- Reader displays results



## Ordering Information

Description	Configuration	Catalog Number	CPT Code
Acucy Influenza A&B Test Kit	25 Test (plus 2 for QC)	1010	Flu A 87804QW Flu B 87804QW-59
Acucy Influenza A&B Control Kit***	5 Pos A/5 Pos B	1011	N/A
Acucy System-Reader and Printer	1 each	1039	N/A
USB Memory Drive****	1 each	1021	N/A
Calibration Device****	1 each	1031	N/A
Printer Roll****	1 each	1019	N/A

\*\*\*Controls included with test kit but may be ordered separately

\*\*\*\*Included with Acucy System but may be ordered separately

### THE AMERICAS

SEKISUI Diagnostics, LLC  
One Wall Street  
Burlington, MA 01803  
Phone: 800 332 1042  
Fax: 800 762 6311  
Email: [questions@sekisui-dx.com](mailto:questions@sekisui-dx.com)

### INTERNATIONAL

SEKISUI Diagnostics (UK) Limited  
Liphook Way, Allington  
Maidstone, Kent, ME16 0LQ, UK  
Phone: +44 1622 607800  
Fax: +44 1622 607801  
Email: [info@sekisui-dx.com](mailto:info@sekisui-dx.com)



Because every result matters™

[sekisuidiagnostics.com](http://sekisuidiagnostics.com)