



POINT-OF-CARE

osom[®]

Strep A Test



OSOM[®] Strep A Test

FOR THE QUALITATIVE DETECTION OF GROUP A STREPTOCOCCAL ANTIGEN

Features and Benefits

- **Accurate** – High Sensitivity. High Specificity
- **Preventative** – Aids in differential diagnosis of Strep A, preventing the unnecessary use of antibiotics
- **Fast and Simple Workflow** – Easy-to-read two-color results, with results in 5 minutes
- **Convenient** – CLIA-waived with no additional equipment required
- **Exceptional Customer Service** – Online training modules and technical support staffed by certified medical technologists/professionals

Performance Characteristics


Sensitivity/Specificity	Sample Type	Limit of Detection	Storage Temperature	Time to Result
96% / 98%*	Throat Swab	<10 colonies**	15-30°C / 59-86°F	5 Minutes

*Compared to Single Swab Culture


**Refer to IFU



SENSITIVITY

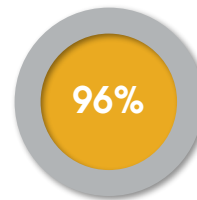

SPECIFICITY

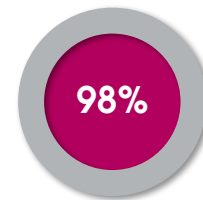


OSOM® Strep A Test




Streppy

SIMPLE TO USE / RESULTS IN MINUTES


1 JUST BEFORE TESTING, ADD 3 DROPS REAGENT 1 (PINK) AND 3 DROPS REAGENT 2 TO THE TEST TUBE (THE SOLUTION SHOULD TURN LIGHT YELLOW)

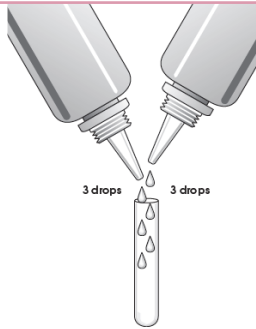


2 INSERT SWAB AND MIX TEN (10) TIMES. LET STAND FOR 1 MINUTE

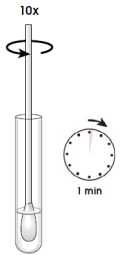

3 SQUEEZE THE SIDE OF THE TUBE AS SWAB IS REMOVED FROM TUBE

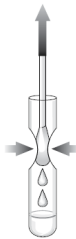

4 PLACE TEST STICK INTO EXTRACTED SAMPLE INSERT AND READ RESULTS AT 5 MINUTES

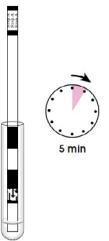

INTERPRETATION OF RESULTS

POSITIVE



NEGATIVE



INVALID



Ordering Information

Description	Configuration	Catalog Number	CPT Code (US only)
OSOM STREP A TEST KIT	50 tests per Kit	141	87880QW
OSOM STREP A TEST KIT (CE MARKED)	50 tests per Kit	141E	N/A
OSOM STREP A TEST KIT (CE MARKED)	20 tests per Kit	141E-20	N/A

To learn more about all of our OSOM Tests scan the code or go to osomtests.com



THE AMERICAS

SEKISUI Diagnostics, LLC
One Wall Street
Burlington, MA 01803
Phone: 800 332 1042
Fax: 800 762 6311

Email: questions@sekisui-dx.com

INTERNATIONAL

SEKISUI Diagnostics (UK) Limited
Liphook Way, Allington
Maidstone, Kent, ME16 0LQ, UK
Phone: +44 1622 607800
Fax: +44 1622 607801

Email: info@sekisui-dx.com



SEKISUI
DIAGNOSTICS

Because every result matters™

sekisuidiagnostics.com