



Meet the Future

Quick. Simple. Mess-Free.


Say goodbye to double-dipping with a mess-free applicator that sends varnish from brush-to-teeth with a quick twist and easy press.

YOUNG
varnish pen[®]
Fluoride Varnish Application System

3 Quick Steps to Mess-Free Varnish!



- 1 With slight downward pressure twist the color dial in the direction of the arrow until flush with the handle.

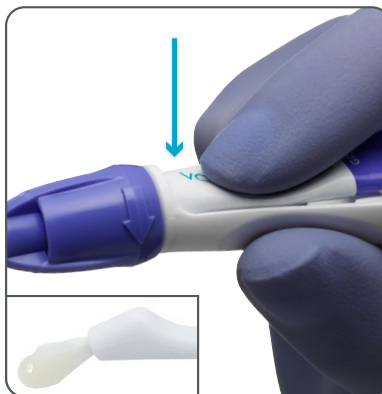

- 2 Gently press the tube until varnish appears on the bristles.



- 3 Apply varnish to teeth in one continuous stroke. No double-dipping!



DID YOU KNOW? VARNISH PEN IS RECYCLABLE!

Simply toss the tip and recycle the handle and tube.